VALO™ X Curing Light

Product Description

With its broadband spectrum, VALO™ X curing light is designed to polymerize all light-cured products in the wavelength range of 380–515 nm per ISO 10650.

The VALO X curing light can be used in a corded or cordless configuration using the Ultradent VALO X rechargeable batteries or provided VALO X cord adapter. The curing light is designed to rest in a standard dental unit bracket or can be custom-mounted using the VALO surface mounting bracket included with the kit.

VALO X curing light Product Components:

- 1 – VALO X Curing Light
- 1 – VALO X Cord Adapter
- 2 – Ultradent VALO X Li-ion 14/65 Batteries
- 1 – Ultradent VALO X Battery Charger with medical grade 9VDC AC Power Supply
- 1 - VALO X barrier sleeve sample pack
- 1 – VALO handheld light shield
- 1 – VALO surface mounting bracket with double stick adhesive tape
- 1 – VALO X curing light Instructions for Use
- 1 – VALO X Interproximal accessory lens
- 1 – VALO X TransLume accessory lens
- 1 – VALO X White/Black light diffuser accessory lens
- 1 – VALO X ProxiCure Ball accessory lens
- 1 - VALO X PointCure accessory lens
- 1 - VALO X Accessory Lens Use brochure

Overview of Controls:

Top button	Activates LED light (single button press)
	Cycles between curing modes (1-second button hold)
Bottom button	Activates LED light (single button press)
	Cycles between diagnostic aid modes (1 second button hold)
To cycle to diagnostic aid modes from curing modes, press and hold the bottom button for 1 second. To cycle back to curing modes from diagnostic aid modes, press and hold the top button for 1 second.	

Alternatively, to cycle between diagnostic aid modes and curing modes using the AF (accelerometer function), see section 8 (stepwise instructions).

Quick Start Guide for Indicators and Alerts:

Mode Indicators on VALO X curing light	State or function
Blue	Curing modes
White and Violet	Diagnostic aid modes
Flashing Green	Sleep Mode

Alert Indicators on VALO X curing light	State or function
Red Flashing	Low battery
Red Steady	Replace/recharge battery
Amber	Temporary LED over temperature protection

Other equipment: Battery Charger	State
Solid Amber	Charging
Solid Green	Charge Complete
Both off	Standby

For all products described, carefully read and understand all instructions and SDS information prior to use.

Indications for Use/Intended Purpose

VALO X curing light is a source of illumination for curing photo-activated dental restorative materials and adhesives. It is also intended to provide illumination to aid in visualization during oral procedures. VALO X curing light accessory/diffuser lenses are not intended for complete cure of photo-activated materials and adhesives.

Contraindications

- No known contraindications have been identified for this product.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- CAUTION Possibly hazardous optical radiation is emitted from this product. DO NOT look directly into the curing light output. Patient, clinician, and assistants should always use amber-colored eye protection when curing light is in use.
- To prevent the risk of electric shock, no modification of this equipment is allowed. Use only the included Ultradent VALO X power supply and plug adapters. If these components are damaged, do not use and call Ultradent Customer Service to order a replacement.
- The VALO X curing light is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. Use of the VALO X curing light adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the VALO X curing light and the other equipment should be observed to verify that they are operating normally. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the VALO X curing light.
- Use only authorized accessories, cables, charger, batteries, and power supplies to prevent improper operation, increased electromagnetic emissions or decreased electromagnetic immunity.

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the VALO X curing light. Otherwise, degradation of the performance of this equipment could result.

- To avoid the risk of electrical fire associated with handling of batteries:
 - DO NOT autoclave or spray battery, battery contacts, charger, or AC power supply with liquid of any kind. If corrosion appears on the contacts of the charger, call Ultradent Customer Service to order a replacement.
 - DO NOT charge batteries around flammable materials
 - DO NOT keep charger in clinical operatory
 - DO NOT charge non-rechargeable batteries/cells
 - DO NOT recharge Ultradent batteries with a non-Ultradent charger
- To avoid the risk of injury, DO NOT use batteries that are corroded (rust), dented, emit an odor or fluids, have a torn or missing wrapping, or are otherwise damaged. Call Ultradent Customer Service to order replacement batteries/cells.
- To prevent the risk of thermal irritation or injury, avoid back-to-back curing cycles and do not expose oral soft tissues at close proximity for more than 10 seconds in Standard Power mode or 5 seconds in Xtra Power mode. If longer curing times are required, use multiple shorter curing cycles or use a dual-cure product to avoid heating soft tissue.
- Use caution when treating patients who suffer from adverse photobiological reactions or sensitivities, patients who are undergoing chemotherapy treatment, or patients being treated with photosensitizing medication.
- This unit may be susceptible to strong magnetic or static electric fields, which could disrupt the programming. If you suspect this has occurred, unplug the unit momentarily and then re-plug it into the outlet.
- DO NOT wipe down the curing light with caustic or abrasive cleaners, autoclave, or immerse in any kind of ultrasonic bath, disinfectant, cleaning solution, or liquid. Failure to follow included processing instructions may render curing light inoperable.
- To avoid damaging the equipment, DO NOT insert fingers, instruments, or other objects into the battery/cell compartment of the curing light.
- To avoid damaging the equipment, DO NOT attempt to clean the electrical contacts, or any part of the battery/cell compartment. Call Ultradent Customer Service if there is a concern.
- To help prevent cross-contamination and help keep dental composite material from adhering to the surface of the lens and wand body, an Ultradent approved barrier sleeve must be used over the VALO X curing light with each use. Barrier sleeves are intended for single-patient use.
- To reduce the risk of corrosion, remove barrier sleeve after use.
- To reduce the risk of under-cured resins, do not use curing light if lens is damaged.
- To prevent the risk of product damage, do not autoclave or dry heat sterilize the VALO X Accessories.
- DO NOT use any of the VALO X Accessories for complete/final cure. Remove all curing accessory lenses after use and perform a complete/final cure without the use of accessories.
- DO NOT use illumination/visualization accessory lenses for curing dental restorative materials and adhesives
- WARNING: This product can expose you to substances which are known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov

Stepwise Instructions

Preparation

1. Prior to each use, place a new barrier sleeve over the curing light, minimizing wrinkles over the lens for best results.
2. Select desired curing mode or diagnostic aid mode. (See below for information on modes and how to cycle through each mode)
3. For optimal results when curing, position curing light centered over and as close to resin as possible without contacting the surface.

Curing Mode	Power ¹ (mW)	Irradiance ¹ (mW/cm ²)	Total Exposure Time (Seconds)	Energy (Joules)	Button Indicator
Standard Power	1,350	1,100	10	13.5	Steady Blue
Xtra Power -	2,700	2,200	5	13.5	Pulsing Blue

¹ Nominal value.

*Xtra Power has a triple pulse with a 0.5 second auto rest between pulses. Please note, there is a 2-second cooling period after using Xtra Power mode.

Changing between curing modes using "drum tap" motion or button push:

Drum Tap: VALO X curing light is equipped with an accelerometer function allowing the user to cycle through curing modes via "drum tap" motion. Use a downward tap motion to activate/change curing mode (see visual aid below).

- Steady Blue Button Indicator = Standard Power
- Pulsing Blue Button Indicator = Xtra Power

Button Push – hold **top** button for 1 second to cycle through 2 curing modes

- Steady Blue Button Indicator = Standard Power
- Pulsing Blue Button Indicator = Xtra Power

Diagnostic Aid Modes

Diagnostic Mode	Power ¹ (mW)	Irradiance ¹ (mW/cm ²)	Total Exposure Time (Seconds)	Button Indicator
Black Light	522	425	60	Pulsing Violet
Whit Light	282	230	60	Pulsing White

² Nominal value, without accessory lens attached.

Changing between diagnostic aid modes using "wave" motion or button push:

Wave: VALO X curing light is equipped with an accelerometer function allowing the user to cycle through diagnostic modes via

"wave" motion. Use a sideways tap motion to activate/change diagnostic aid mode (see visual aid below).


- Violet Button Indicator = Black Light Diagnostic Aid Mode
- White Button Indicator = White Light Diagnostic Aid Mode

Button Push: Push and hold **bottom** button for 1 second to cycle through 2 diagnostic aid modes.

Violet Button Indicator = Black Light Diagnostic Aid Mode – 1 cycle at 1 minute - Turn off with either button press or wave tap motion
White Button Indicator = White Light Diagnostic Aid Mode – 1 cycle at 1 minute - Turn off with either button press or wave tap motion

Accessory/diffuser lenses are attached magnetically over the curing light lens. Place barrier sleeve on curing light prior to attaching accessory lens.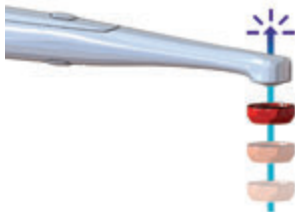

NOTE: Upon holding either button, you will get an audible indication at 1 second and 10 seconds.

Sleep Mode

Push and hold both buttons for 1 second to put in sleep mode. Upon holding both top and bottom buttons you will get an audible indication when entering sleep mode. When entering sleep mode, you must continue holding until the audible tone is complete. When you enter the sleep mode by pressing both buttons, the VALO X curing light will not wake until either button is pressed.

VALO X Accessories

Accessory	Mode	Description
PointCure Lens	Curing Modes	Augments the VALO X curing light to polymerize composite through a translucent prosthetic.
ProxiCure Ball Lens	Curing Mode	Augments the VALO X curing light to polymerize composite and help shape contact area matrix of an interproximal restoration.
Diffuser Lens	White Light Diagnostic Aid Mode	Augments the VALO X curing light to provide a visual aid for accurate color/shade comparison or whenever natural light is needed.
	Black Light Diagnostic Aid Mode	Augments the VALO X curing light to provide visualization of fluorescing chemicals in dental resins.
Interproximal Lens	White Light Diagnostic Aid Mode	Augments the VALO X curing light in visualization of teeth and dental prostheses.
TransLume Lens	Curing or Diagnostic Aid Modes	Augments the VALO X curing light in visualization by providing longer wavelength light to transilluminate teeth and dental prostheses.

Preparation

- Position accessory lens near the VALO X curing light lens allowing the magnet to snap the accessory lens into place over the barrier sleeve.

Use – PointCure Lens

- Curing Mode - The PointCure Lens concentrates the light into a 2.5 mm aperture. This is ideal for point curing (tacking) veneers and all porcelain crowns.
- Curing Mode - Point curing (tacking) translucent prosthesis: Point cure prosthesis according to manufacturer guidelines of the luting material, material thickness, shade, opacity of material, and professional judgement. Clean up the uncured excess luting material around the margins, remove the accessory, then cure the entire restoration. Recommendations: Veneer - point cure the facial/buccal center of prosthesis. Crown - point cure facial/buccal and lingual center of prosthesis.

Use – ProxiCure Ball Lens

- Curing Modes - Supports the matrix and aids in convex proximal contact before and during partial light polymerization. Avoid entrapping ProxiCure Ball Lens in polymerized material. Remove accessory lens and cure the entire restoration for complete cure. Recommendation: Determine curing mode and time as per composite manufacturer's instructions.

Use – Diffuser Lens

- Black Light Diagnostic Aid Mode - Use to aid in visualization of fluorescent particles in various dental resins. Gives 60 seconds of illumination.
- White Light Diagnostic Aid Mode - Visual aid for whenever the dental professional may need a source of natural light, e.g., determining shade. Gives 60 seconds of illumination.

Use – Interproximal Lens

- White Light Diagnostic Aid Mode - Transillumination of teeth: use to aid in visualization of fractures, cracks, or defects in teeth by illuminating the teeth from the lingual and observing shadows.


Use – TransLume Lens

- White Light Diagnostic Aid Mode or Curing Mode - Neon-colored lens, which looks like the PointCure Lens, provides longer wavelength light to transilluminate the teeth to aid in visualization of cracks, fractures, defects, etc. The White Light Diagnostic Aid Mode gives 60 seconds of illumination.

Powering the VALO X Curing Light

- **Charging Batteries/Cells** - Remove cap at the base of the curing light with a counter-clockwise turn. Insert dead battery/cells

in the provided Ultradent VALO X Battery Charger with VALO X Power Supply in the orientation shown in the charger. Battery/cells will be fully charged in 1–3 hours.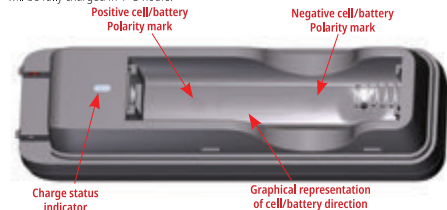

Charge State Description	Charge Status Indicator	Voltage Condition	Charge Current
Pre-Charge	Solid Amber	2.86Vdc to 3.15Vdc	95mA
Bulk Charge	Solid Amber	3.15Vdc to 4.20Vdc	950mA
Charge Complete	Solid Green	>4.20Vdc	N/A
Standby	Both off	N/A	N/A

- **Cord Adapter** - Remove cap at the base of the curing light with a counter-clockwise turn. Remove battery/cell. Insert cord adapter into battery/cell compartment, fully seat adapter and turn clockwise to lock into place. Position VALO X Power Supply so it can be disconnected without difficulty from MAINS power.

Mounting Bracket Instructions

- 1) Bracket should be mounted to a flat, oil-free surface.
- 2) Clean surface with rubbing alcohol.
- 3) Peel backing off the bracket's adhesive tape.
- 4) Position bracket so the curing light lifts upward when removed. Press firmly into place.
- 5) If running the curing light on battery/cell power, the curing light should be oriented downward in the bracket. If running the curing light on the VALO X Cord Adapter power, the curing light should be oriented upward in the bracket.

Maintenance

Repair

User-Performed Maintenance

- 1) End of life is determined by wear and damage due to use. Inspect all components for damage prior to use and contact authorized service personnel to repair damage.
- 2) Routinely check the lens for cured dental resins. If necessary, use a plastic or stainless-steel dental instrument to carefully remove any adhered resin.
- 3) Routinely check the output in Standard Power mode. Light meters differ greatly and are designed for specific light guide tips and lenses. NOTE: the true numeric output will be skewed due to the inaccuracy of common light meters and the custom LED pack in

the curing light; however, contact authorized service personnel if a decrease in output is observed when measuring with the same meter.

Manufacturer Repair

- 1) Repairs are only to be performed by authorized service personnel. Ultradent will provide service personnel with documentation to perform repairs.

Warranty

Ultradent Products, Inc. ("Ultradent") warrants that this product shall, for a period of 5 years from the date of purchase, when operated according to the operating instructions included with the product, (i) conform in all material respects to the specifications set forth in Ultradent's documentation accompanying the product; and (ii) be free from defects in material and workmanship. This limited warranty is nontransferable and applies solely to the original purchaser and does not extend to subsequent owners of the product. This limited warranty does not cover any accessory components such as, but not limited to, batteries, chargers, adapters, or adaptive lenses. This limited warranty is void if the product is damaged due to negligence, abuse, misuse, accident, modification, tampering, alteration, or failure to follow the applicable instructions for use. To qualify under this limited warranty, proof of purchase (e.g., sales receipt or similar documentation) must be submitted to Ultradent along with the defective product.

Ultradent will either repair or replace defective products, in its sole discretion, that fall under this warranty. In no event shall Ultradent's liability for the product exceed the purchase price paid by the purchaser. Under no circumstances shall Ultradent be liable for any indirect, incidental, unforeseen, special, or consequential damages arising out of or in connection with the use of this product.

Processing

Disinfection Reprocessing Instructions

VALO Curing Light Processing Instructions

Disinfection Reprocessing Instructions	
Initial treatment at the time of use	Can be disinfected in the event the device becomes contaminated during use. A new Ultradent approved disposable barrier sleeve must be used for each patient.
Preparation before disinfection	Use a 70% isopropyl alcohol (IPA) wipe to thoroughly wipe the entire surface. Discard soiled wipe.
Rinsing	None
Drying	Allow to completely air dry at ambient temperature.
Maintenance, inspection, and testing	Visually inspect that the device is not damaged and is free of debris. If debris is present, repeat preparation step with a new wipe.
Packaging	No packaging required for disinfection.

Disinfection	Wipe the entire surface and keep wet for the exposure time while paying special attention to cracks, crevices, seams, and hard-to-reach areas. Use additional wipes as necessary to keep the surface wet.	
	Disinfectant	Exposure Time
	70% IPA	4 minutes
	70% - 90% ethyl alcohol wipe	6 minutes
Drying	Allow to completely air dry at ambient temperature.	
Storage	Store in a clean and dry location. To prevent moisture buildup, do not store in a barrier sleeve.	
Additional information	This procedure was validated for intermediate-level disinfection by an independent and accredited laboratory. Note: Use of cleaning agents other than what is prescribed above, such as bleach or cleaners containing phosphoric acid, may cause damage to equipment. DO NOT attempt to clean the electrical contacts, or any part of the battery/cell compartment.	

EN VALO Accessories Processing Instructions

Disinfection Reprocessing Instructions				
Initial treatment at the time of use	Remove accessory prior to disinfection between each use.			
Preparation before disinfection	Remove any cured dental composites from accessory lenses prior to cleaning.			
Cleaning	Place in an ultrasonic cleaner with 1 part detergent to 9 parts warm water using the product listed below:			
	<table border="1"> <tr> <td>Detergent</td> <td>Cleaning Time</td> </tr> <tr> <td>Henry Schein General Purpose Cleaner or equivalent product</td> <td>2-10 minutes</td> </tr> </table>	Detergent	Cleaning Time	Henry Schein General Purpose Cleaner or equivalent product
Detergent	Cleaning Time			
Henry Schein General Purpose Cleaner or equivalent product	2-10 minutes			
Rinsing	Rinse accessory with warm water to remove detergent for 1–2 minutes.			
Drying	Dry with gauze. Air dry 30 minutes.			
Maintenance, inspection and testing	Visually inspect that the accessory is not damaged and free of debris. If cracked or damaged, discontinue use. If debris is present, repeat cleaning process.			
Packaging	No packaging required for disinfection (sterilization packages can be used for storage after disinfection).			
Disinfection	Remove from storage packaging (if applicable). Submerge accessory in disinfection solution as per manufacturer's instructions listed below:			
	<table border="1"> <tr> <td>Disinfectant</td> <td>Exposure Time</td> </tr> <tr> <td>Cidex® OPA Solution or equivalent product</td> <td>12 minutes</td> </tr> </table>	Disinfectant	Exposure Time	Cidex® OPA Solution or equivalent product
Disinfectant	Exposure Time			
Cidex® OPA Solution or equivalent product	12 minutes			
Rinsing	Rinse per disinfectant manufacturer's instructions listed below: 1. Following removal from CIDEX® OPA Solution, thoroughly rinse the accessory by immersing it completely in a large volume (e.g., 2 gallons) of water. 2. Keep the accessory totally immersed for a minimum of 1 minute in duration. 3. Remove the accessory and discard the rinse water. Always use fresh volumes of water for each rinse. Do not reuse the water for rinsing or any other purpose. 4. Repeat the procedure two (2) additional times, for a total of three (3) rinses, with large volumes of fresh water to remove CIDEX® OPA Solution residues. Residues may cause serious side effects. SEE WARNINGS. THREE (3) SEPARATE, LARGE VOLUME WATER IMMERSION RINSES ARE REQUIRED. Additionally, rinse in water in ultrasonic cleaner for 5 minutes.			
Drying	Dry with gauze. Air dry 30 minutes.			
Storage	Store in a clean and dry location.			
Additional information	This procedure was validated by an independent and accredited laboratory.			

Storage and Disposal

If storing the curing light for periods longer than 2 weeks, or packing it for travel, always remove the batteries. If batteries are left in the unit for long periods of time without recharging, they may become nonfunctional or un-chargeable. Do not store batteries in temperatures over 60°C (140°F) or in direct sunlight.

Storage and Transport Conditions:

- Temperature: +10°C to +40°C (+50°F to +104°F)
- Relative Humidity: 10% to 95%
- Ambient Pressure: 500 hPa to 1060 hPa


When disposing of electronic waste, (i.e. devices chargers, batteries and power supplies), follow local waste and recycling guidelines.

Technical Considerations

Accessories

Item	CE Information		
VALO Batteries VALO X Power Supply VALO X Battery Charger		Manufactured by: GlobTek, Inc. 186 Veterans Drive, Northvale, NJ 07647 Made in China	Distributed by: Ultradent Products Inc 505 West Ultradent Drive (10200 South) South Jordan, UT 84095 USA

Technical Information/Data

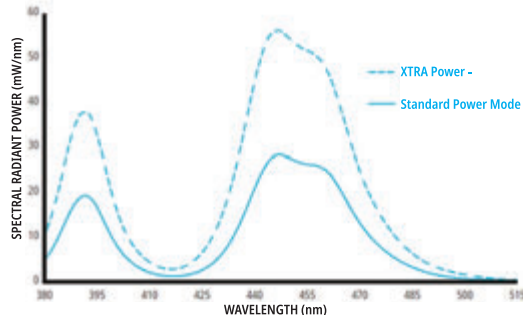Attribute	Information/Specification	
Lens	Diameter 12.5 mm = 1.2271 cm ² (122.71 mm ²) PointCure Lens: Diameter 2.5 mm ProxiCure Ball Lens: Diameter 2.1 mm	
Wavelength range	Utilizable wavelength range: 380–515 nm Peak wavelengths: 380–420 nm and 420–515 nm	
Light Intensity Table	Standard Power - 1,100 mW/cm ² (+10%) Xtra Power - 2,200 mW/cm ² (+10%) (Conforms to ISO 10650 when measured with a Gigahertz spectrum analyzer) Black Light - 230-648 mW/cm ² White Light - 160-300 mW/cm ² NOTE: PointCure and ProxiCure Ball Lenses focus the light energy into a smaller aperture, thus increasing the light intensity above calibrated values.	
VALO X Curing Light	The curing light with a barrier sleeve is an applied part. Ratings: IEC 60601-1 (Safety), IEC 60601-1-2 (EMC), Ingress Protection Rating: IP54	Weight: With battery: 4.8 oz. (136 grams) Without battery: 3.8 oz. (108 grams) With cord adapter: 5.6 oz. (158 grams) Dimension (battery configuration): 8.9 x 0.83 x 0.83 inches, (226 x 21 x 21 millimeters)
VALO X Power Supply	Output - 9VDC at 2A Input - 100VAC to 240VAC, 50-60 Hz Ultradent P/N 4952 - VALO X Power Supply with international plug inserts	Ratings: IEC 60601-1 (Safety) Cord Length - 6 feet (1.8 meters) VALO X Power Supply is a Medical Grade Class II power supply and provides isolation from MAINS power
VALO X Battery Charger	VALO X Lithium Ion (Li-Ion) Battery Charger: Output - 4.2VDC at 950mA bulk charge current Input - 9VDC at 2A (700mA minimum) Automatic shut off when fully charged Auto-detection of defective cell Protections: Overcharge, Short-circuit, Reverse polarity Amber LED - Charging Green LED - Fully charged Charging time: 1–3 hours Ratings: IEC 60601-1 (Safety), CE, WEEE	
VALO X Li-ion 14/65 Batteries	Protected, Rechargeable, Li-Ion Battery 11MR14/65 3.7V, 900mAh 3.33Wh 3.7V, 950mAh Max. 3.52Wh Ratings: WEEE, IEC 62133-2 Cells are rated for >500 charge/discharge cycles before hitting 70% of its capacity. It is advisable to replace batteries every 2-3 years.	
Operating Conditions	Temperature: +10°C to +32°C (+50°F to +90°F) Relative Humidity: 10% to 95% Ambient Pressure: 700 hPa to 1060 hPa	

Curing Mode Duty Cycle:	At maximum ambient temperature (32°C) 10 seconds ON and 60 seconds OFF. Note - Duty cycle is defined by primary function of device (polymerize resin) using Xtra Power mode, which creates highest amount of thermal energy. Maximum Device temperature when following duty cycle: ≤45°C
-------------------------	--

Troubleshooting

If the solutions suggested below do not rectify the problem, please call Ultradent at 800.552.5512. Outside the United States, call your Ultradent distributor or dental dealer.

Problem	Possible Solutions
Light will not turn on	<ul style="list-style-type: none"> • Press the Power Button to wake from Sleep Mode • Confirm battery/cell is fully charged • Check that cell is correctly inserted into the unit • If Amber LED alert appears, the curing light has reached its internal temperature safety limit. Allow the curing light to cool down. • If Red or Amber LED alert flashes continuously, call Ultradent Customer Service for repair
Light does not stay on for desired time	<ul style="list-style-type: none"> • Check mode indicator for correct device setting • Check the Low Power indicator for cell charge status • Check that a fresh cell is properly inserted into the unit
Light is not curing resins properly	<ul style="list-style-type: none"> • Check lens for residual cured resins/composites • Using proper amber UV eye protection, verify the LED lights are working • Check power level with light meter. Refer to maintenance section for specific instructions on using light meter. • Check expiration date on curing resin • Ensure proper technique is being followed according to manufacturer recommendations
Battery will not charge	<ul style="list-style-type: none"> • Make sure the cell is inserted into the charger in the correct orientation and allow cell to fully charge for up to 3 hours • If Amber lights on the charger do not change to Green, call Ultradent Customer Service to order replacement cells and/or charger • If neither Green nor Amber lights on the charger are visible when a battery is inserted into charger, call Ultradent Customer Service
Charger does not charge battery	<ul style="list-style-type: none"> • Make sure charger is plugged in and AC power supply is plugged into a working power outlet • If neither Green nor Amber lights on the charger are visible, call Ultradent Customer Service

Miscellaneous Information

Electromagnetic Environment Guidance –

- VALO X curing light is suitable for use in all establishments, including domestic establishments, that are directly connected to the public low-voltage power supply network (AC MAINS).
- VALO X curing light can operate from two different power sources.
 - Battery powered: When VALO X curing light uses the battery, it is not affected by EMI, RF, electric transients, surges, voltage dips, shorts, interruptions, or variations from AC MAINS power.
 - Adaptor powered: VALO X curing light can use a Globtek medical grade 9VDC adaptor to operate from AC MAINS power ranging from 100VAC – 240VAC. It provides limited EMI, RF, brown out, and surge suppression.
- VALO X curing light uses electrical and electromagnetic energy only for their internal functions. Therefore, any RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
- The VALO X curing light is not equipped with a DC power port, Patient Coupling Ports, or Signals Input/Output ports, therefore testing to any of these ports is not applicable. Only compliance with applicable configurations are listed.

Table 5.1 Summary of Emission Tests: VALO X curing light complies with CISPR 11 Group 1, Class B for radiated emissions.

Port	Test Description	Frequency Range (MHz)	Result
AC Power	Conducted Emissions (Hot Lead to Ground)	0.15 to 30	Complied
AC Power	Conducted Emissions (Neutral Lead to Ground)	0.15 to 30	Complied
Enclosure	Radiated Emissions (Vertical Polarity)	30 to 1000	Complied
Enclosure	Radiated Emissions (Horizontal Polarity)	30 to 1000	Complied

Table 5.2 Summary of Immunity Tests (Enclosure Port and AC Power Port):

Basic Standard	Environmental Phenomena	Result
EN 61000-4-2	Electrostatic Discharge	Complied
EN 61000-4-3	Radio Frequency Electromagnetic Field	Complied
EN 61000-4-3	Proximity fields from RF wireless communication equipment	Complied
EN 61000-4-4	Electrical fast transient/burst	Complied
EN 61000-4-5	Lightning Surge (line-line)	Complied
EN 61000-4-6	Radio-frequency continuous conducted	Complied
EN 61000-4-11	Voltage Dips and Interruptions	Complied

Table 5.3 Summary of Tests (EN 60601-3-2 and EN 61000-3-3; AC Power Port)

Basic Standard	Environmental Phenomena	Result
EN 61000-4-2	Harmonic Current Measurement	Complied
EN 61000-4-3	Voltage Flicker Measurement	Complied

FCC Regulatory Statement

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference, and
- 2) This device must accept any interference received, including interference that may cause undesired operation.

Warning

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Interference

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Re-orient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

ISED Regulatory Statement (includes French)

This radio transmitter has been approved by Industry Canada to operate with the antenna types listed with the maximum permissible gain indicated. Antenna types not included in this list, having a gain greater than the maximum gain indicated for that type, are strictly prohibited for use with this device.

This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions:

- 1) This device may not cause interference, and
- 2) This device must accept any interference, including interference that may cause undesired operation of the device.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to ICES-003. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Re-orient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Report any serious incident to the manufacturer and the competent authority.

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	Warning to read and following instructions and pay attention to specific warnings and cautions related to the use of a curing light.
	Refer to instruction manual/booklet.
	Compliance with EU regulations for safety, electrical and electronic interference and immunity.
	Authorized representative.
	Authorized representative.
	Compliance with Medical Class B electrical safety.
	WEEE compliant: Recycle; do not discard improperly.

	Class II Equipment. Wall power supply for the charger is a Class II medical grade, double insulated supply.
	Manufactured by Ultradent in accordance with GMP and ISO 13485 practices.
	Date of Manufacture
	Do not re-use to avoid cross contamination
	Upper and lower temperature limitations
	Upper and lower relative humidity limitations: Store or transport the device in areas within the relative humidity range of 10 to 95%.
	Follow local waste and recycling guidelines when disposing of Lithium Ion batteries.

	EN - Upper and lower atmospheric pressure limitations: Store or transport the device in areas within the atmospheric pressure range of 1060 to 500 hPa.
	Keep Dry: Cargoes bearing this symbol must be protected from excessive humidity and must be stored under cover.
	Serial Number
	Nemko Electrical Safety and EMC
	Ingress Protection Rating
	Home symbol indicates that the equipment is only for indoor use and not for industrial purposes.
	Indicates that the product contains certain toxic or hazardous substances or elements, and can be used safely during its environmental protection use period.

	EN - For professional use only DE - Nur zur Anwendung durch den Zahnarzt FR - Pour usage professionnel seulement NL - Alleen voor professioneel gebruik IT - Solo per uso professionale ES - Solo para uso profesional PT - Apenas para utilização profissional SV-Endast för professionell användning DA - Kun til professionel brug FI - Vain ammattikäyttöön EL - Για επαγγελματική χρήση μόνο CS - Pouze pro odborné použití, BG - Само за професионално ползване HR - Samo za profesionalnu uporabu ET - Ainult professionaalseks kasutuseks HU - Kizárólag professzionális felhasználásra GA - Le haghaidh úsáide gairmiúla amháin LV - Tikai profesionālai lietošanai LT - Tik profesionaliam naudojimui MT - Għall-użu professjonali biss RO - Numai pentru uz profesional SK - Len na profesionálne použitie SL - Samo za profesionalno uporabo NO - Kun for profesjonell bruk PL - Wyrób wyłącznie do użytku profesjonalnego. RU - Только для профессионального использования TR - Sadece profesyonel kullanim için AR - لاستخدام من قبل متخصصين فقط TH - สำหรับการใช้งานโดยมืออาชีพเท่านั้น KO - 전문가 전용 ZH - 仅供专业人士使用 JA - 専門家のみが使用できます
---	--

Manufactured by:

 **Ultradent Products, Inc.**
505 West Ultradent Drive (10200 South)
South Jordan, Utah 84095 USA
Manufactured in the USA from globally sourced materials

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Australian Sponsor:
Ultradent Australia Pty Ltd
P.O. Box 111 Coogee, NSW 2034
1800 29 09 29 | www.ultradent.com.au

NOTES:

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**Ultradent Products GmbH**
**Qunique GmbH**